GASTROTUSS®

SCIROPPO ANTIREFLUSSO

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GASTROTUSS®

ANTI-REFLUX SYRUP

Please read right through this leaflet before you start using this product.

1. What Gastrotuss® Syrup is and what is used for

Gastrotuss® Syrup is a medical device with mechanical action, indicated for the treatment of gastroesophageal and/or laryngopharyngeal reflux to reduce the symptoms related to it such as: acid or basic regurgitation, pyrosis (heartburn), globus pharyngeus (the lump or foreign body sensation in the throat), rhinopharyngolaryngeal inflammation, belching, abdominal bloating, cough, dysphonia (impairment in the voice), and all cases of disorders that have gastroesophageal and/or laryngopharyngeal reflux as their main or concomitant cause. Gastrotuss® Syrup is an adjuvant in the re-epithelization processes of the gastroesophageal and/or laryngopharyngeal mucosa. Gastrotuss® Syrup reduces the time of permanence of the gastric content in the stomach.

For oral use.

2. What you need to know before you take Gastrotuss® Syrup

- Do not take Gastrotuss® Syrup in case of known hypersensitivity to any component of the product (listed in Section 6).
- Talk to your doctor before taking Gastrotuss® Syrup if you are pregnant or breastfeeding.
- The product may be more or less viscous, without any deterioration of its quality.
- The presence of natural substances can have an impact on the organoleptic characteristics of the product but possible variations in color and/or taste do not affect its quality.

3. How to take Gastrotuss® Syrup

Bottle

Shake well before use

Adults and children aged 12 years and over:

Take a 20 ml dose after main meals and at bedtime or according to different medical advice.

Monodose Sachet

Adults and children aged 12 years and over:

Take a 20 ml sachet after main meals and at bedtime or according to different medical advice.

4. Warnings

- In case of adverse reaction, discontinue use and consult a doctor.
- Do not exceed the recommended doses, follow the directions.
- It is recommended to take Gastrotuss Syrup at least 1 hour before or after any medicines.
- Do not use this product if the original packaging appears not intact or there is evidence of tampering (for example, if it is re-labeled). Tampering and/or non-intactness of the packaging could compromise the safety requirements of the product.
- Do not use this product after the expiration date shown on the pack.
- The expiry date refers to the intact and properly stored product.

5. How to store Gastrotuss® Syrup

- Keep out of the reach and sight of children.
- Close the bottle tightly after use.
- Do not store above 30 °C

6. Further information

Ingredients: Magnesium Alginate, Simethicone, Fructose, Xanthan Gum, Honey, D-Panthenol, Fluid Extracts of Althea Officinalis and Papaver Rhoeas, Zinc Oxide, Sodium Bicarbonate, Sodium Hydroxide, Sodium Methyl p-Hydroxybenzoate, Sodium Propyl p-Hydroxybenzoate, Natural Flavours, Erythrosine (E127), Purified Water.

Gluten and Lactose free.

Presentations

- Bottle containing 500 ml or 200 ml, with a measuring spoon.
- Pack containing 25 bags (20 ml each).

Unit destined for the Italian market.

via Laurentina Km. 26,700 - 00071 Pomezia (Rome) - ITALY - www.dmg.it