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Confezione:

Flacone da 20 ml 

Ear Drops facilitating the removal of cerumen plug

Please read right through this leaflet before you start using this product.

1. What CUE® CAP Ear Drops is and what is used for

CUE® CAP is an otologic solution to soften and facilitate the removal of the cerumen from the external auditory canal. It is also indicated before ENT visits or prior to local drug treatments, and to keep clean the ear canal of hearing aid wearers. CUE® CAP contains a balanced mixture of propylene glycol and natural glycols with solvent capacity. Propolis extract carries out a cleansing and softening action, allantoin contributes to alleviate the itching and any possible discomfort of the external auditory canal, with a soothing action. CUE® CAP can also be used for an effective prevention of cerumen impaction.

2. What you need to know before you use CUE® CAP Ear Drops

Do not use this product in case of:

- Known hypersensitivity or allergy to any component of the product (listed in Section 6).
- Confirmed perforation of the tympanic membrane.

3. How to use CUE® CAP Ear Drops

For preparation to the removal of the cerumen plug:

- Apply 3-4 drops twice a day for at least 5 days or according to medical advice.
- CUE® CAP can also be used in the presence of small skin lesions.

4. Warnings

- In case of adverse reactions, discontinue use and consult a doctor.
- Use the utmost caution at instillation to avoid trauma caused by accidental contact with the dropper.
- Do not use this product if the original packaging appears not intact or there is evidence of tampering (for example, if it is re-labeled). Tampering and/or non-intactness of the packaging could compromise the safety requirements of the product.
- Do not use this product after the expiration date shown on the pack. The expiry date refers to the intact and properly stored product.
- Use the product for one patient only.
- Avoid contact with eyes.
- Do not swallow.
- Do not apply the product in conjunction with the application of medicinal or other topical products.

5. How to store CUE® CAP Ear Drops

- Keep out of the reach and sight of children.
- Close the bottle tightly after use.
- Do not store above 25° C.

6. Further information

Ingredients: Propylene glycol, Propanediol, Propolis Extract, Allantoin, PEG-40 Hydrogenated Castor Oil, PPG-26 Buteth 26, Trideceth-9, Boric acid, Sodium tetraborate, Purified water.

Presentation:

20 ml  bottle