CuE®

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CuE[®]

Otologic Solution

Please read right through this leaflet before you start using this product.

1. What CuE[®] Otologic Solution is and what is used for

CuE[®] Otologic Solution is a medical device with mechanical action. With its lubricant, emollient, protective and soothing properties, is indicated both in adults and children for preventing and helping the treatment of the irritations and inflammations of the outer ear canal like mycosis, eczema, dermatitis, external otitis ("swimmer's ear"). In case of environmental factors (warm-wet climate, air-conditioning, chlorinated or sea water) or external pathogens, the oily extracts and the vegetable glycerol contained in CuE[®] help, thanks to their emollient and protective action, to prevent the otitis and the inflammation of the outer ear canal, making the product highly recommended, for example, when practicing water sports in both the sea and the swimming pool.

2. What you need to know before you use CuE[®] Otologic Solution

Do not use this product in case of

- Known hypersensitivity to any component of the product (listed in Section 6).
- Confirmed or suspected perforation of the tympanic membrane.

3. How to use CuE[®] Otologic Solution

- Administer 3-4 drops twice a day for 5-7 days or according to doctor's opinion.
- During instillation, make sure to avoid any injury due to accidental contact with the dropper.

4. Warnings

- In case of adverse reactions, discontinue use and consult a doctor.
- Do not use this product in case of known hypersensitivity or individual allergy to any component of the product (listed in Section 6)
- Consult a doctor in case of confirmed or suspected injuries of the outer ear canal and/or the tympanic membrane.
- A slight transient itching sensation can occur over a 15 minute period after the application, if applied to injured skin.
- Do not use this product if the original packaging appears not intact or there is evidence of tampering (for example, if it is re-labeled). Tampering and/or non-intactness of the packaging could compromise the safety requirements of the product.
- Do not use the product after the expiration date shown on the pack. The expiry date refers to the intact and properly stored product.
- Use the product for one patient only.
- Avoid contact with eyes.
- Do not swallow.
- Do not apply the product in conjunction with the application of medicinal or other topical products.

5. How to store CuE[®] Otologic Solution

- Keep out of the reach and sight of children.
- Close the bottle tightly after use.
- Do not store above 25 °C.

6. Further information

Ingredients: Propylene Glycol, Glycerol, Climbazol, Dipotassium Glycyrrhizinate, Acetic Acid, Allantoin, Oily Extracts (Origanum Vulgare, Thymus Vulgaris, Cinnamomum Zeylancum, Olea Europaea, Rosmarinus Officinalis, Mentha Piperita, Lavandula Angustifolia, Hydrastis Canadensis, Citrus Limonum), purified water.

Presentation: 15 ml  bottle.

via Laurentina Km. 26,700 - 00071 Pomezia (Rome) - ITALY - www.dmgit.com