Elastar®

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Acido ialuronico sale sodico Cross-linkato e Carbossimetil β -glucano

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Elastar®

Ophthalmic Solution

Cross-linked Hyaluronic Acid Sodium Salt and Carboxymethyl β -glucan
Preservative-free

Please read right through this leaflet before you start using this product.

1. What Elastar® Ophthalmic Solution is and what is used for

Elastar® Ophthalmic Solution is a preservative-free tear substitute. It contains cross-linked hyaluronic acid sodium salt and carboxymethyl β -glucan.

Elastar® Ophthalmic Solution is used for the protection, hydration and lubrication of the eye surface.

The cross-linked hyaluronic acid sodium salt, due to its reticular structure, guarantees a greater hydration and viscoelasticity, a high stability of the tear film and remains for a long time on the ocular surface.

The carboxymethyl β -glucan, in addition to increasing the lubricating ability of the product, counteracts the oxidative stress caused by free radicals and supports the reorganization of the corneal cells.

The OSD (Ophthalmic Squeeze Dispenser) technology allows the innovative delivery of an absolutely sterile preservative-free ophthalmic solution.

Elastar® Ophthalmic Solution is indicated for the treatment of the symptomatology (dryness, foreign body sensation, etc.) associated with

- Dry eye
- Mechanical stress: use of contact lenses, surgical procedures, etc.
- Systemic diseases: Sjögren's syndrome, rheumatoid arthritis, liver disease, etc.
- Hormonal disorders: menopause, hormone replacement therapy, etc.
- Environmental stress: air conditioning, cigarette smoke, wind, etc.
- Visual stress: use of video terminal.
- Use of drugs: antidepressants, beta-blockers, prostaglandins, etc.

This product can be used by wearers of contact lenses.

2. What you need to know before you use Elastar® Ophthalmic Solution

Do not use Elastar® Ophthalmic Solution if you are allergic or in case of known hypersensitivity to any component of the product (listed in Section 6).

3. How to use Elastar® Ophthalmic Solution

- You do not need to remove contact lenses before using Elastar® Ophthalmic Solution.
- Wash your hands before applying the product.
- Instill 1-2 drops in each eye 2-3 times a day or according to medical advice.
- Make sure that the dropper does not come into contact with the eye.

4. Warnings

- In case of adverse reaction, discontinue use and consult a doctor.
- Ocular use, do not swallow.
- Make sure that the bottle is intact before use.
- Do not use this product after the expiration date shown on the pack. The expiry date refers to the intact and properly stored product.

5. How to store Elastar® Ophthalmic Solution

- Keep out of the reach and sight of children.
- Discard the bottle 90 days after opening.
- Close the bottle tightly after use and store at room temperature and away from heat sources.

6. Further information

Ingredients: Cross-linked Hyaluronic Acid Sodium Salt 0.15%, Carboxymethyl β -glucan 0.3%, pH 7.2-isotonic buffered solution.

Presentation

10 ml bottle with OSD dispenser

Italdevice srl

via Laurentina Km. 26,700
00071 Pomezia (Rome) - Italy
www.italdevice.com

Rev. 03/2016

Distributore:

Via Laurentina Km 26,700
Pomezia-Roma-Italy
www.dmgit.com

Elastar® is a Medical Device according to
Council Directive 93/42/EEC

MADE IN ITALY

STERILE A

CE
1370

2004019 4-19/01/2021