IT

Unguento Barriera Emostatico

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EN

Haemostatic Barrier Ointment

Please read right through this leaflet before you start using this product.

1. What Emofix® is and what is used for

Emofix® is an haemostatic ointment with barrier function indicated for the containment and prevention of cutaneous and mucosal haemorrhagic episodes such as, for example, essential, even slight, or traumatic epistaxis, small skin wounds, etc.

2. What you need to know before you use Emofix®

Do not use this product in case of known hypersensitivity to any component of the product (listed in Section 6).

3. How to use Emofix®

To contain an haemorrhagic episode: apply a sufficient quantity of the ointment to cover the injured area.

To prevent the recurrence of epistaxis: apply the product twice a day for at least two weeks after the acute episode, or according medical advice.

The use of the applicator cannula, included in the pack, is recommended in case of application on the nasal mucosa.

Clean carefully the cannula after each application and avoid sharing it.

4. Warnings

- In case of adverse reactions stop treatment and consult a doctor.
- Burning sensation, nasal itching and rhinorrhea can appear in the first days of the treatment. They usually tend to disappear in the course of the therapy. If this does not happen, stop treatment and consult a doctor.
- Do not use this product if the original packaging appears not intact or there is evidence of tampering (for example, if it is re-labeled). Tampering and/or non-intactness of the packaging could compromise the safety requirements of the product.
- Do not use this product after the expiration date shown on the pack. The expiry date refers to the intact and properly stored product.

5. How to store Emofix®

- Keep out of the reach and sight of children.
- Close the tube tightly after use.
- Do not store above 30 °C. If stored at different temperature the product can vary in compactness, without any decline in quality. By simply maintaining it at room temperature, Emofix® will recover the original solidity.

6. Further information

Ingredients: Mix of saturated fatty acids, yeast protein extract, phosphatidylcholine, tocopheryl acetate, beeswax, soy oil, stearyl alcohol, calcium, potassium, magnesium chlorides, glyceryl monostearate, methyl and propyl p-hydroxybenzoate, BHT.

Presentation

30g Tube with applicator cannula.

Unit destined for the Italian market.