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Via Laurentina Km 26,700
Pomezia-Roma-Italy - www.dmgit.com

Rev. 12/2016

Ophthalmic Solution Preservative-free

Please read right through this leaflet before you start using this product.

1. What Enoxoftal® Ophthalmic Solution is and what is used for

Enoxoftal® Ophthalmic Solution is a medical device with phlogolytic activity (it inhibits the evolutionary processes of the phlogosis). It is indicated for the treatment of the symptoms (itching, foreign body sensation, redness, etc...) associated with inflammatory conditions of the ocular surface, such as ocular discomfort syndrome, blepharitis and conjunctivitis, in children and adults.

Enoxoftal® Ophthalmic Solution, thanks to a specific thermosensitive copolymer able to form a gel when entering in contact with the eye surface, provides prolonged moisturizing and lubricating effects. The OSD (Ophthalmic Squeeze Dispenser) technology allows the innovative delivery of an absolutely sterile preservative-free ophthalmic solution.

2. What you need to know before you use Enoxoftal® Ophthalmic Solution

Do not use Enoxoftal® Ophthalmic Solution in case of known hypersensitivity to any component of the product (listed in Section 6).

3. How to use Enoxoftal® Ophthalmic Solution

- Wash your hands before applying the product.
- Instill 1-2 drops in each eye 2-3 times a day or according to medical advice.
- Make sure that the dropper does not come into contact with the eye.

4. Warnings

- Cases of corneal calcification associated with the use of ophthalmic solutions containing phosphates were very rarely found in patients with severe damages of the ocular surface. Consult your doctor before use this product in case of band keratopathy or deep calcium deposition in the cornea.
- In case of adverse reaction, discontinue use and consult a doctor.
- A slight transient itching sensation can occur shortly after the instillation.
- Ocular use, do not swallow.
- Make sure that the bottle is intact before use.
- Do not use this product after the expiration date shown on the pack. The expiry date refers to the intact and properly stored product.

5. How to store Enoxoftal® Ophthalmic Solution

- Keep out of the reach and sight of children.
- Discard the bottle 60 days after opening.
- Close the bottle tightly after use
- Store at room temperature and away from heat sources.

6. Further information

Ingredients: Enoxolone glucuronide, Hyaluronic Acid Sodium Salt 0.10%, copolymer PEG/PPG, potassium hydroxide, sodium phosphate monobasic, sodium phosphate dibasic, sodium chloride, purified water.

Presentation

10 ml bottle with OSD dispenser

Enoxoftal® is a Medical Device according to Council Directive 93/42/EEC