GastrOftal®

IT

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GastrOftal[®]

EN

Ophthalmic Solution
Preservative-Free

Please read right through this leaflet before you start using this product.

1. What GastrOftal[®] Ophthalmic Solution is and what is used for

GastrOftal[®] Ophthalmic Solution is a Medical Device indicated for the treatment of tear dysfunction symptoms (irritation, itching, tearing, foreign body sensation, eye fatigue sensation, etc. ...) either transitory or associated with chronic diseases.

GastrOftal[®] Ophthalmic Solution, thanks to the synergistic action of hyaluronic acid, magnesium alginate and extract of *Camellia sinensis* (green tea), hydrates, lubricates, and protects the ocular surface from both external factors (e.g. ultraviolet rays, wind, dust, smog, contact lenses) and endogenous substances, such as the proteolytic enzymes (pepsin), which are associated with gastroesophageal reflux disease.

The OSD (Ophthalmic Squeeze Dispenser) technology allows the innovative delivery of an absolutely sterile preservative-free ophthalmic solution.

2. What you need to know before you use GastrOftal[®] Ophthalmic Solution

Do not use GastrOftal[®] Ophthalmic Solution if you are allergic or in case of known hypersensitivity to any component of the product (listed in Section 6).

3. How to use GastrOftal[®] Ophthalmic Solution

- Wash your hands before applying the product.
- Instill 1-2 drops in each eye 2-3 times a day or according to medical advice.
- Make sure that the dropper does not come into contact with the eye.
- You do not need to remove contact lenses before using GastrOftal[®] Ophthalmic Solution.

4. Warnings

- In case of adverse reaction, discontinue use and consult a doctor.
- Ocular use, do not swallow.
- For pediatric age follow your doctor instructions.
- Make sure that the bottle is intact before use.
- Do not use this product after the expiration date shown on the pack. The expiry date refers to the intact and properly stored product.

5. How to store GastrOftal[®] Ophthalmic Solution

- Keep out of the reach and sight of children.
- Discard the bottle 90 days after opening.
- Close the bottle tightly after use.
- Store at room temperature and away from heat sources.
- Do not dispose of the bottle into the environment after use.

6. Further information

Ingredients: Hyaluronic Acid Sodium Salt, Magnesium Alginate, Glycerine, Extract of *Camellia sinensis* (green tea), pH 7.6-buffered solution.

Presentation

10 ml bottle with OSD dispenser

Medical Device according to Council Directive 93/42/EEC