GASTROTUSS[®] *light*

IT

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EN

HYPOCALORIC ANTI-REFLUX SYRUP

Please read right through this leaflet before you start using this product.

1. What Gastrotuss[®] Light is and what is used for

Gastrotuss[®] Light is a medical device with mechanical action, indicated for the treatment of gastroesophageal and/or laryngopharyngeal reflux to reduce the symptoms related to it such as: acid or basic regurgitation, pyrosis (heartburn), globus pharyngeus (the lump or foreign body sensation in the throat), rhinopharyngolaryngeal inflammation, belching, abdominal bloating, cough, dysphonia (impairment in the voice), and all cases of disorders that have gastroesophageal and/or laryngopharyngeal reflux as their main or concomitant cause. Gastrotuss[®] Light is an adjuvant in the re-epithelization processes of the gastroesophageal and/or laryngopharyngeal mucosa.

Gastrotuss[®] Light reduces the time of permanence of the gastric content in the stomach.

Gastrotuss[®] Light is low in calories and sodium, therefore being indicated for the treatment of reflux even in overweight/obese patients, diabetic patients and in all cases where a low-calorie and/or low sodium diet is needed.

For oral use.

2. What you need to know before you take Gastrotuss[®] Light

- Do not take Gastrotuss[®] Light in case of known hypersensitivity to any component of the product (listed in Section 6).
- Talk to your doctor before taking Gastrotuss[®] Light if you are pregnant or breastfeeding.
- The product may be more or less viscous, without any deterioration of its quality.

3. How to take Gastrotuss[®] Light

Shake well before use

Children aged 3 to under 12 years:

Take a 10-20 ml dose after main meals and at bedtime or according to different medical advice.

Adults and children aged 12 years and over:

Take a 20 ml dose after main meals and at bedtime or according to different medical advice.

4. Warnings

- In case of adverse reaction, discontinue use and consult a doctor.
- Do not exceed the recommended doses, follow the directions.
- It is recommended to take Gastrotuss[®] Light at least 1 hour before or after any medicines.
- Do not use this product if the original packaging appears not intact or there is evidence of tampering (for example, if it is re-labeled). Tampering and/or non-intactness of the packaging could compromise the safety requirements of the product.
- Do not use this product after the expiration date shown on the pack.
- The expiry date refers to the intact and properly stored product.

5. How to store Gastrotuss[®] Light

- Keep out of the reach and sight of children.
- Close the bottle tightly after use.
- Store at room temperature and away from heat sources.

6. Further information

Ingredients: Sorbitol, Magnesium Alginate, Simethicone, Potassium Bicarbonate, Xanthan Gum, Methyl p-Hydroxybenzoate, EDTA Disodium Salt, Potassium Hydroxide, Propyl p-Hydroxybenzoate, D-Panthenol, Sucralose, Flavouring, Purified Water.

Gluten and Lactose free.

Presentation

500 ml bottle with a measuring spoon.

Unit destined for the Italian market.

via Laurentina Km. 26,700 - 00071 Pomezia (Rome) - ITALY - www.dmgit.com