GRIP STOP®

IT

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GRIP STOP®

Protective and Unplugging Nasal Drops

Please read right through this leaflet before you start using this product.

1. What GRIP STOP® Nasal Drops is and what is used for

GRIP STOP® Nasal Drops is a medical device with mechanical action, indicated for the treatment of the symptoms associated with cold inflammatory conditions, also of bacterial origin, of the nasal cavities. GRIP STOP® Nasal Drops helps re-establish the normal nasal respiration and tubal ventilation, in adults and children.

2. What you need to know before you use GRIP STOP® Nasal Drops

Do not use this product in case of known hypersensitivity to any component of the product (listed in Section 6).

3. How to use GRIP STOP® Nasal Drops

Recommended dosage: administer two drops into each nostril three-four times a day or according to doctor's opinion. GRIP STOP® Nasal Drops can also be used in case of little injuries of the nasal mucosa or of the skin around the nose.

4. Warnings

- In case of adverse reactions stop treatment and consult a doctor.
- Do not swallow.
- Use the product for one patient only.
- Avoid contact with eyes.
- Make sure that the container and/or the bottle is intact before use.
- Do not use this product after the expiration date shown on the pack. The expiry date refers to the intact and properly stored product.

5. How to store GRIP STOP® Nasal Drops

- Keep out of the reach and sight of children.
- Close the bottle tightly after use.
- Do not store above 30° C.

6. Further information

Ingredients: D-panthenol, Dipotassium glycyrrhizinate, Hydroxypropyl methylcellulose, Lactoferrin, Carboxymethyl β -glucan, Chlorobutanol, EDTA disodium salt, Sodium N-hydroxymethyl glycinate, Polysorbate 20, Natural flavourings, Buffer solution pH 7.0.

Presentation

15 ml bottle with dropper cap.

Unit destined for the Italian market.

via Laurentina Km. 26,700 - 00071 Pomezia (Rome) - ITALY - www.dmgit.com