GRIP STOP®

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GRIP STOP®

EN

Protective and Unplugging Nasal Spray

Please read right through this leaflet before you start using this product.

1. What GRIP STOP® Nasal Spray is and what is used for

GRIP STOP® nasal spray is a medical device with mechanical action, indicated for the treatment of the symptoms associated with cold inflammatory conditions, also of bacterial origin, of the nasal cavities. GRIP STOP® Nasal Spray helps re-establish the normal nasal respiration and tubal ventilation, in adults and children.

2. What you need to know before you use GRIP STOP® Nasal Spray

Do not use this product in case of known hypersensitivity to any component of the product (listed in Section 6).

3. How to use GRIP STOP® Nasal Spray

Remove the protective cap, place your thumb at the bottom of the bottle, your index and middle fingers on the flaps of the pump and push them down. Then release. Repeat this operation until the first spray is delivered. Tilt your head forward slightly and after having breathed out slowly, administer the product to the nostril. Breathe in and, using a finger on your other hand, close the nostril on the side not receiving the product.

Recommended dosage: two puffs for each nostril twice a day or according to doctor's opinion.

GRIP STOP® Nasal Spray can also be used in case of little injuries of the nasal mucosa or of the skin around the nose.

4. Warnings

- In case of adverse reactions stop treatment and consult a doctor.
- Do not swallow.
- Use the product for one patient only.
- Avoid contact with eyes.
- Make sure that the container and/or the bottle is intact before use.
- Do not use this product after the expiration date shown on the pack. The expiry date refers to the intact and properly stored product.

5. How to store GRIP STOP® Nasal Spray

- Keep out of the reach and sight of children.
- Close the bottle tightly after use.
- Do not store above 30 °C.

6. Further information

Ingredients: D-panthenol, Dipotassium glycyrrhizinate, Lactoferrin, Carboxymethyl β -glucan, Chlorobutanol, EDTA disodium salt, Sodium N-hydroxymethyl glycinate, Polysorbate 20, Natural flavourings, Buffer solution pH 7.0.

Presentation

20 ml bottle with mechanical nebulizer.

Unit destined for the Italian market.

via Laurentina Km. 26,700 - 00071 Pomezia (Rome) - ITALY - www.dmgit.com

Rev. 02/2019 30FG071 4-18/01/2021