



Nasal spray with carboximethyl β -glucan

Please read right through this leaflet before you start using this product.

1. What IMMUWASH® Nasal Spray is and what is used for

IMMUWASH® Nasal Spray is a medical device containing isotonic solution with carboximethyl β -glucan. It acts by mechanical action as a cleanser, moisturizer, lubricator and pH re-balancer of nasal fluid. Useful for prevention and reduction of stagnation and post-surgery complications; for the alterations of the mucus film due to environmental stress (smog, cigarette smoke, etc.), foreign bodies (pollen, dust etc.) or temporary pathologies like sinusitis, adenoiditis, polyposis, rhinitis with or without microbial disease. Indicated for the symptomatic treatment of the otitis media and of the rhinogenous hypacusia, in synergy with other treatments, to maintain nasal cavity cleaned, helping tubal ventilation. Indicated for the symptomatic treatment of the chronic rhinosinusitis with or without nasal polyposis. During the post-operative recovery, IMMUWASH Nasal Spray moisturizes, lubricates and supports the healing process.

2. What you need to know before you use IMMUWASH® Nasal Spray

Do not use this product in case of known hypersensitivity to any component of the product (listed in Section 6).

3. How to use IMMUWASH® Nasal Spray

Remove the protective cap. Place your thumb at the bottom of the bottle, your index and middle fingers on the flaps and push them down. Then release. Repeat this operation until the first spray is delivered. Blow your nose before using the product, then tilt your head forward slightly and after having breathed out slowly, administer the product to the nostril. Breathe in, and using a finger on your other hand, close the nostril on the side not receiving the product. Spray, if necessary several times, and wait for a short interval allowing the solution to act before blowing your nose. Repeat the administration as required. IMMUWASH® Nasal Spray can also be used in case of little injuries of the nasal mucosa or of the skin around the nose.

4. Warnings

- In case of adverse reactions stop treatment and consult a doctor.
- Do not swallow.
- Use the product for one patient only.
- Avoid contact with eyes.
- Do not use this product if the original packaging appears not intact or there is evidence of tampering (for example, if it is re-labeled). Tampering and/or non-intactness of the packaging could compromise the safety requirements of the product.
- Discard the bottle 90 days after opening
- Do not use this product after the expiration date shown on the pack. The expiry date refers to the intact and properly stored product.
- Do not apply the product in conjunction with the application of medicinal or other topical products.

5. How to store IMMUWASH® Nasal Spray

- Keep out of the reach and sight of children.
- Close the bottle tightly after use.
- Do not store above 25° C.
- Do not discharge the container in the environment after use.

6. Further information

Ingredients: Carboxymethyl β -glucan, EDTA disodium salt, Sodium benzoate, N-hydroxymethylglycinate, Isotonic buffered solution pH 7.20.

Presentation

50 ml bottle with 100 mcl mechanical spray pump unit.

Each bottle can deliver 500 sprays.

Without propellants.

Unit destined for the Italian market



DRUGS MINERALS AND GENERICS ITALIA

