NARIVENT®

Unguento nasale antiedemigeno

Indicazioni:

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Contiene cannula applicatrice.

20 g e

MADE IN ITALY
Fabbricante/Manufacturer

DRUGS MINERALS AND GENERICS ITALIA

NARIVENT®

Antiedemigen Nasal Ointment

Indications:

Medical device with an antiedemigen and phlogolitic osmotic action. Used for the treatment of specific (allergic) or non-specific rhinitis in order to prevent nasal polyp formations.

Indicated to reduce the nasal obstruction in patients affected by polyposis.

Indicated for prophylaxis to prevent relapses after surgical treatment of nasal polyposis.

Composition:

Mannitol, petrolatum, mineral oil, milk albumin, dipotassium glycyrrhizinate, chlorobutanol, tocopheryl acetate, BHT, EO mint.

Usage:

It is recommended to apply the ointment for two annual cycles lasting 60-90 days each or according to your physician's advice.

Apply the preparation inside the nostrils, gently massaging the wings of the nose, twice a day, in the morning and evening, using the enclosed applicator if necessary.

Adverse reactions:

No known adverse reactions have been reported.

Cautions:

- **Do not use in case of known hypersensitivity to any of the product's constituents;**
- In case of adverse reaction, stop the treatment and consult a physician;
- Close tightly after use;
- Never use the product after the expiry date indicated on the package;
- Store away from heat sources and keep out of the reach of children;
- Do not ingest;
- If the applicator is necessary, particular care should be taken to avoid traumas.
- Wash the intranasal applicator carefully after each use.

It contains an intranasal applicator.

20 g e

MADE IN ITALY
Fabbricante/Manufacturer

DRUGS MINERALS AND GENERICS ITALIA

20FG220-01 18-22/01/2021

Rev. 2 of 24/05/2013

Via Laurentina, km 26,700 - 00071 Pomezia, Rome - Italy - www.dmgit.com