NEOEDEMALT[®]

Unguento antiedemigeno

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EN

Antiedema ointment

Please read right through this leaflet before you start using this product.

1. What NEOEDEMALT[®] is and what is used for

NEOEDEMALT[®] is a medical device with osmotic action for treatment, in adults and children, of edematous states of inflammatory and traumatic origin such as sprains, bruises or muscular lesions.

2. What you need to know before you use NEOEDEMALT[®]

Do not use NEOEDEMALT[®] in case of known hypersensitivity to any component of the product (listed in Section 6).

3. How to use NEOEDEMALT[®]

Apply the product on the affected area by performing a gentle massage.

To enhance the effect of the ointment, it is possible to make a compress by applying a uniform layer of product and proceeding to the protective wrap.

Repeat the operation every 6-8 hours or following the doctor's advice, if different.

Before proceeding any further application, it is advisable to remove the product residues with a dry gauze.

4. Warnings

- In case of adverse reactions stop treatment and consult a doctor.
 - Do not swallow.
 - Avoid contact with eyes.
 - Make sure that the container and/or the tube is intact before use.
 - Do not use this product after the expiration date shown on the pack.
- The expiry date refers to the intact and properly stored product.

5. How to store NEOEDEMALT[®]

- Keep out of the reach and sight of children.
- Tightly close the tube after use.
- Do not store above 25° C.

6. Further information

Ingredients: Petrolatum, mineral oil, mannitol, Arnica Montana extract, albumin from milk, vitamin E acetate, mint e.o., BHT, chlorobutanol.

Presentation

100 ml tube.

Via Laurentina Km. 26,700 - 00071 Pomezia (Rome) - ITALY - www.dmgit.com

Rev. 12/2021 30FG142 3-23/12/2021