PepsinO®

IT

Spray Orale

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EN

Oral Spray

Please read right through this leaflet before you start using this product

1. What PEPSINO® is and what is used for

PEPSINO® is a medical device with mechanical action, indicated for the reduction of laryngopharyngeal symptoms of gastroesophageal reflux such as: globus pharyngeus (the foreign body sensation in the throat), cough, need to scrape the throat, laryngopharyngeal dryness or irritation, etc.

PEPSINO® can also lubricate the laryngopharyngeal mucosa, helping the re-epithelization processes.

For oral topical use.

2. What you need to know before you take PEPSINO®

- Do not take PEPSINO® in case of known hypersensitivity to any component of the product (listed in Section 6).
- Do not take PEPSINO® if you are pregnant or breastfeeding.

3. How to take PEPSINO®

- Lift the cannula for spraying.
- When first used, point the spray away from the oral cavity and prime it by pumping a few times until a fine spray appears.
- Introduce the cannula in the mouth and direct the aspersion to the throat, pressing firmly the dosing valve.
- Apply two puffs three/four times a day until the symptoms disappear or following the doctor's advice.

4. Warnings

- In the persistence of symptoms, despite treatment, consult a doctor.
- In case of adverse reaction, discontinue use and consult a doctor.
- Do not apply the product simultaneously with the application of medicines or other topical products.
- Avoid contact with eyes.
- Do not exceed the recommended dose and follow the directions.
- Do not use this product if the original packaging appears not intact or there is evidence of tampering (for example, if it is re-labeled). Tampering and/or non-intactness of the packaging could compromise the safety requirements of the product.
- Do not use this product after the expiration date shown on the pack. The expiry date refers to the intact and properly stored product.
- Use the product for one patient only.

5. How to store PEPSINO®

- Keep out of the reach and sight of children.
- Do not store above 30 °C.

6. Further information

Ingredients: Purified Water, Fructose, Fluid Extract of Camellia sinensis (Green Tea), Magnesium alginate, Sodium Benzoate, Potassium Sorbate, Hyaluronic Acid sodium salt, EDTA disodium salt, Potassium Citrate, Citric Acid, Mint Flavouring.

Presentation

- 30 ml oral spray bottle.

Unit destined for the Italian market.

via Laurentina Km. 26,700 - 00071 Pomezia (Rome) - ITALY - www.dmgit.com

Rev. 03/2020 30FG069 2-18/01/2021