Rinocross®

IT

Spray Nasale con Acido ialuronico Cross-linkato, D-Pantenolo, Biotina, Vit. A e Vit. E

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Rinocross®

EN

Nasal Spray with cross-linked Hyaluronic Acid, D-Panthenol, Biotin, Vitamin A and Vitamin E

Please read right through this leaflet before you start using this product.

1. What RINOCROSS® Nasal Spray is and what is used for

RINOCROSS® Nasal Spray is a medical device that lubricates and maintains the hydration and the elasticity of the nasal mucosa, helping the re-epithelization process.

Cross-linked Hyaluronic Acid (0,1%) and D-Panthenol (1%) guarantee greater hydration and viscoelasticity of the nasal mucus. Biotin, Vitamin A and Vitamin E contribute to the normal trophism of the mucous membranes.

RINOCROSS® Nasal Spray is indicated:

- In case of dry and/or atrophic nasal mucosa caused by particular environmental conditions or treatments (dust, irritating substances, decongestant nasal sprays, oxygen therapy, etc.)
- To support the treatment of the sinonasal diseases (rhinosinusitis, allergic rhinitis, rhinitis medicamentosa, chronic atrophic rhinitis, etc.)
- In the post-surgical treatment of nasal and sinonasal diseases, to promote the restoration of the normal function of the mucosa.

2. What you need to know before you use RINOCROSS® Nasal Spray

Do not use this product in case of known hypersensitivity to any component of the product (listed in Section 6).

3. How to use RINOCROSS® Nasal Spray

- Remove the protective sealed cap. Place your thumb at the bottom of the bottle, your index and middle fingers on the flaps and push them down. Then release. Repeat this operation until the first spray is delivered. Tilt your head forward slightly and after having breathed out slowly, administer the product to the nostril. Breathe in and, using a finger on your other hand, close the nostril on the side not receiving the product.
- Recommended dosage: two puffs for each nostril twice a day or according to doctor's opinion.
- Rinocross® Nasal Spray can also be used in case of little injuries of the nasal mucosa or of the skin around the nostrils.

4. Warnings

- In case of adverse reactions stop treatment and consult a doctor.
- Do not swallow.
- Avoid promiscuous use.
- Avoid contact with eyes.
- Make sure that the container and/or the bottle is intact before use.
- Do not use this product after the expiration date shown on the pack. The expiry date refers to the intact and properly stored product.

5. How to store RINOCROSS® Nasal Spray

- Keep out of the reach and sight of children.
- Close the bottle tightly after use.
- Do not store above 30 °C.

6. Further information

Ingredients: D-Panthenol, Vitamin E acetate, Vitamin A palmitate, Biotin, Hydrogenated Castor Oil 40 oe, cross-linked Hyaluronic Acid, EDTA disodium salt, Sodium N-hydroxymethylglycinate, Potassium sorbate, Natural flavouring, Isotonic buffered solution at pH 7,2

Presentation: 20 ml bottle

Rev. 01/2019

via Laurentina Km. 26,700 - 00071 Pomezia (Rome) - ITALY - www.dmgit.com

30FCG93 3-26/03/2021