RINOPANTEINA®

Spray Nasale con Vitamina A ed E

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RINOPANTEINA®

EN

Nasal Spray with Vitamin A and Vitamin E

Please read right through this leaflet before you start using this product.

1. What RINOPANTEINA® Nasal Spray is and what is used for

Rinopanteina® Nasal Spray is a medical device which lubricates and maintains the hydration of the nasal mucosa, helping the reepithelization process.

Rinopanteina® Nasal Spray is indicated in case of dry and/or atrophic nasal mucosa caused by particular environmental conditions, nasal sprays, aerosol therapy or temporary pathologies (i.e. rhinitis medicamentosa or allergic rhinitis, ozena).

2. What you need to know before you use RINOPANTEINA® Nasal Spray

Do not use this product in case of known hypersensitivity to any component of the product (listed in Section 6).

3. How to use RINOPANTEINA® Nasal Spray

- Remove the protective cap. Place your thumb at the bottom of the bottle, your index and middle fingers on the flaps and push them down. Then release. Repeat this operation until the first spray is delivered. Tilt your head forward slightly and after having breathed out slowly, administer the product to the nostril. Breathe in, and using a finger on your other hand, close the nostril on the side not receiving the product.
- Recommended dosage: two puffs for each nostril twice a day or according to doctor's opinion.
- Rinopanteina® Nasal Spray can also be used in case of little injuries of the nasal mucosa or of the skin around the nostrils.

4. Warnings

- In case of adverse reactions stop treatment and consult a doctor.
- Do not swallow.
- Avoid promiscuous use.
- Avoid contact with eyes.
- Make sure that the container and/or the bottle is intact before use.
- Do not use this product after the expiration date shown on the pack. The expiry date refers to the intact and properly stored product.

5. How to store RINOPANTEINA® Nasal Spray

- Keep out of the reach and sight of children.
- Close the bottle tightly after use.
- Do not store above 30 °C

6. Further information

Ingredients: D-Panthenol , Vitamin E acetate, Vitamin A palmitate, Hydrogenated Castor Oil 40 oe, EDTA disodium salt, Sodium N-hydroxymethylglycinate, Potassium sorbate, Xanthan Gum, Natural flavouring, Isotonic buffered solution at pH 7,2.

Presentation

20 ml bottle

Rev. 01/2019

via Laurentina Km. 26,700 - 00071 Pomezia (Rome) - ITALY - www.dmgit.com

30FCG92 4-21/07/2023