RINOPANTEINA®

UNGUENTO E GOCCE NASALI

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EN

OINTMENT AND NASAL DROPS

Please read right through this leaflet before you start using this product.

1. What Rinopanteina[®] is and what is used for

Rinopanteina[®] is a medical device (ointment and nasal drops) which lubricates and maintains the hydration of the nasal mucosa, helping the re-epithelization process.

Rinopanteina[®] is indicated:

- After a surgical operation, to reduce the formation of crusts;
- In case of dry and/or atrophic nasal mucosa caused by particular environmental conditions, nasal sprays, aerosol therapy or temporary pathologies (i.e. rhinitis medicamentosa or allergic rhinitis, ozena);
- In order to reduce the consequences of epistaxis also when caused by a traumatic event;
- In order to facilitate endoscopic examinations and/or tamponade maneuvers.

The ointment is useful for prevention and treatment of cold-induced chapped skin around the nose.

2. What you need to know before you use Rinopanteina[®]

Do not use this product in case of known hypersensitivity to any component of the product (listed in Section 6).

3. How to use Rinopanteina[®]

Ointment: Apply a small quantity of ointment one-three times a day and massage it gently to the inside of each nostril to spread the ointment throughout the nares.

Drops: Shake vigorously the bottle before use and then administer 2-3 drops into each nostril 4-5 times a day.

4. Warnings

- In case of adverse reactions stop treatment and consult a doctor.
- Do not use this product if the original packaging appears not intact or there is evidence of tampering (for example, if it is re-labeled). Tampering and/or non-intactness of the packaging could compromise the safety requirements of the product.
- Do not use this product after the expiration date shown on the pack. The expiry date refers to the intact and properly stored product.
- Close the container tightly after use.

5. How to store Rinopanteina[®]

- Keep out of the reach and sight of children.
- Store the product between 15°C and 25°C.
- Drops: At low temperature the drops can become semi-solid, without any deterioration in product quality. The product will return to its original state by simply shaking and maintaining at an ambient temperature (not inferior to 15°C).

6. Further information

Ointment Ingredients: Vitamin A Palmitate (0.05%), D-Panthenol (5.0%), Hydrolyzed Protein (Avena Sativa extract), Sodium Hyaluronate (0.1%), Ceteareth-20, Sodium Benzoate, BHA, Mineral Oil, Petrolatum, Purified Water.

Drops Ingredients: Vitamin A Palmitate (0.05%), D-Panthenol (0.1%), BHA, Mineral Oil, Petrolatum.

Presentations

Ointment: 10 g tube

Drops: 30 ml bottle

Unit destined for the Italian market.

via Laurentina Km. 26,700 - 00071 Pomezia (Rome) - ITALY - www.dmgit.com

20FGJ157 5-04/08/2021

Rev.09/2019