STOMATOVIS®

Collutorio

IT

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STOMATOVIS®

Mouthwash

Please read right through this leaflet before you start using this product.

1. What STOMATOVIS® Mouthwash is and what is used for

STOMATOVIS® Mouthwash is a medical device indicated for the treatment and prevention of the symptoms associated with stomatitis and gingivitis of different origin. It enhances lubrication and hydration of mucosa of oral cavity.

2. What you need to know before you use STOMATOVIS® Mouthwash

Do not use STOMATOVIS® Mouthwash in case of known hypersensitivity to any component of the product (listed in Section 6).

3. How to use STOMATOVIS® Mouthwash

- Swish an undiluted measuring cup of the product (15-20 ml) in the oral cavity and hold it on your oral mucosa for at least 60 seconds (do not shallow).
- Do not rinse.
- Repeat the application from 1 to 3 times a day.
- Wash carefully the measuring cap after each use and avoid sharing it.

4. Warnings

- In case of adverse reaction, discontinue use and consult a doctor.
- Do not apply the product simultaneously with the application of medicines or other topical products.
- Do not exceed the recommended doses, follow the instructions for use.
- Do not use this product if the original packaging appears not intact or there is evidence of tampering (for example, if it is re-labeled). Tampering and/or non-intactness of the packaging could compromise the safety requirements of the product.
- Do not use this product after the expiration date shown on the pack. The expiry date refers to the intact and properly stored product.
- Due to the ingredients, the product may be opalescent or may show particles in suspension that are dispersed by shaking vigorously without any loss of product quality.
- Do not use the product 90 days after it is opened.

5. How to store STOMATOVIS® Mouthwash

- Keep out of the reach and sight of children.
- Close the bottle tightly after use.
- Do not store above 30°C.

6. Further information

Ingredients: Purified water, xylitol, mallow extract, vegetable glycerol, dipotassium glycyrrhizinate, carboxymethyl β -glucan, sodium bicarbonate, chlorobutanol, PVM/MA copolymer, N-hydroxymethylglycinate sodium salt, PEG-40 hydrogenated castor oil, acesulfame-K, flavouring, green color (E102–E131– E514).

Presentation

200 ml bottle with measuring cup.

Unit destined for the Italian market.

Rev. 11/2021

30FG119-A1 4-22/11/2021

via Laurentina Km. 26,700 - 00071 Pomezia (Rome) - ITALY - www.dmg.it