STOMATOVIS®

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Indicazioni:

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Rev. 1 del 02/03/2010

MADE IN ITALY
Fabbricante/Manufacturer

DRUGS MINERALS AND GENERICS ITALIA

Via Laurentina, km 26,700 - 00071 Pomezia, Rome - Italy - www.dmgit.com

STOMATOVIS®

Protective Stomatological Paste

Indications:

Protective adhesive barrier used for the treatment of the ulcerous states following the aphthous stomatitis.

Composition:

Mineral Oil, Petrolatum, PVM/MA Ca/Na salt, Sodium Carboxymethyl Cellulose, Xylitol, D-Panthenol, Carboxymethyl beta-glucan, Aloe Vera extract, Mallow mucilage, Tocopheryl acetate, Dipotassium glycyrrhizinate, Vitamin A palmitate, Zinc Oxide, Chlorobutanol, Cupric Chlorophyll, Flavouring.

Usage:

Apply a little quantity of the product to cover the interested area using a light pressure, allow to dry for at least twenty seconds avoiding, because of its adhesiveness, the direct contact of the product with other parts of the mouth. Repeat the administration according to need.

Cautions:

- Do not apply the product directly from the tube.
- Do not use in case of Known hypersensitivity to any of the constituents of product.
- In case of adverse reactions stop treatment and consult a doctor.
- Close the tube tightly after use.
- Never use the product after the expiry date.
- Always keep it away from heat sources and out of the reach of children
- Little quantities of the product, if casually ingested, are not harmful.

MADE IN ITALY
Fabbricante/Manufacturer

DRUGS MINERALS AND GENERICS ITALIA