SafloVir®

IT

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EN

Sanitizer Protective Nasal Spray

Please read right through this leaflet before you start using this product.

1. What SafloVir® Nasal Spray is and what is used for

- SafloVir® is a medical device with chelating (sequestering) and lubricating action on the nasal cavities, in adults and children.
- SafloVir®, thanks to the synergic action of its components (Lactoferrin, dipotassium glycyrrhizinate, carboxymethyl beta-glucan, vitamin C and vitamin D), sanitizes and protects the nasal cavities from external agents, even of viral or bacterial origin.

2. What you need to know before you use SafloVir® Nasal Spray

Do not use this product in case of known hypersensitivity to any component of the product (listed in Section 6).

3. How to use SafloVir® Nasal Spray

- Remove the protective sealed cap. Place your thumb at the bottom of the bottle, your index and middle fingers on the flaps and push them down. Then release. Repeat this operation until the first spray is delivered. Tilt your head forward slightly and after having breathed out slowly, administer the product to the nostril. Breathe in and, using a finger on your other hand, close the nostril on the side not receiving the product.
- Recommended dosage: two puffs for each nostril twice a day or according to doctor's opinion.

4. Warnings

- In case of adverse reactions stop treatment and consult a doctor.

- Do not swallow.
- Avoid promiscuous use
- Use during pregnancy and lactation: No specific precautionary measures are necessary. If SafloVir® Nasal Spray is used according to indications, no increased risk is known during pregnancy or lactation.
- Avoid contact with eyes.
- Make sure that the container and/or the bottle is intact before use.
- Do not use this product after the expiration date shown on the pack. The expiry date refers to the intact and properly stored product.

5. How to store SafloVir® Nasal Spray

- Keep out of the reach and sight of children.
- Close the bottle tightly after use.
- Store at a temperature not exceeding 30°C.

6. Further information

Ingredients: Lactoferrin, Dipotassium glycyrrhizinate, Carboxymethylβ-glucan, D-panthenol, Vitamin C (Ascorbic acid), Vitamin D3 (Colecalciferol), Chlorobutanol, EDTA disodium salt, N-hydroxymethylglycinate sodium salt, Polysorbate 20, Flavour, Buffered solution pH 7.20.

Presentation

20 ml bottle with mechanical spray pump.

via Laurentina Km. 26,700 - 00071 Pomezia (Rome) - ITALY - www.dmgit.com

30F5667-A1 3-04/04/2025
Rev. 01 04/2025