TeRsO®

IT

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TeRsO®

EN

Ophthalmic Solution with Osmotic Action

Please read right through this leaflet before you start using this product.

1. What Terso® Ophthalmic Solution is and what is used for

Terso® Ophthalmic Solution is a Medical Device indicated for the osmotic treatment of the edema and the sub-edema of the cornea, adjuvant treatment of the edematous symptoms of glaucoma.

2. What you need to know before you use Terso® Ophthalmic Solution

Do not use Terso® Ophthalmic Solution if you are allergic or in case of known hypersensitivity to any component of the product (listed in Section 6).

3. How to use Terso® Ophthalmic Solution

- Wash your hands before applying the product.
- Instill 1-2 drops in each eye 2-3 times a day or according to medical advice.
- Make sure that the dropper does not come into contact with the eye.

4. Warnings

- In case of adverse reaction, discontinue use and consult a doctor.
- Ocular use, do not swallow.
- Make sure that the bottle is intact before use.

Do not use this product after the expiration date shown on the pack. The expiry date refers to the intact and properly stored product.

5. How to store Terso® Ophthalmic Solution

- Keep out of the reach and sight of children.
- Discard the bottle 60 days after opening.
- Close the bottle tightly after use.
- Do not store above 30 °C.
- Do not dispose of the bottle into the environment after use.

6. Further information

Ingredients: Mannitol, glycerol, EDTA disodium salt, chlorobutanol, hyaluronic acid sodium salt, buffered solution at pH 7,2.

Presentation

8 ml bottle.

Rev. 04/2021

via Laurentina Km. 26,700 - Pomezia (Rome) - ITALIA - www.dmg.it

STERILE A

20FG2 102-A1 11-28/04/2021