Viscoblast®

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EN

Indications:

Moisturizing, lubricating ophthalmic solution useful for the treatment of eye dryness caused by various factors (menopause, diabetes, etc...); helps the re-epithelization process; indicated for the relief of corneal hypersensitivity due to traumatic injuries including post-operative, prolonged video display terminal use and/or sun's rays; photochemical smog.

Composition:

Carboxymethyl β -glucan 0,5%, Sodium N-hydroxymethylglycinate 0,001%, EDTA disodium salt 0,100%, Isotonic buffered solution at pH 7,2 q.s for ml 100.

Usage:

Administer one or two drops into each eye 2-3 times a day or following your physician's advice.

Adverse reactions:

No known adverse reactions have been reported.

Cautions:

- This product will remain sterile until opened;
- Close tightly after use;
- Discard the product 60 days after opening;
- Do not use the product after the expiry date printed on the container;
- Keep out of reach of children;
- Do not ingest;
- Do not use in case of known hypersensitivity to any of the ingredients of the product;
- In case of adverse reactions, discontinue immediately the treatment and consult your physician;
- Avoid contact of the container tip with your eyes, its surrounding structures, your hands or any other surface;
- Do not use the product if the package does not appear intact.

Viscoblast® is a Medical Device according to Council Directive 93/42/EEC

15 ml bottle e